

Features

- ▶ 100% Digital
- ▶ Open-Fit Device
- ▶ AirTouch™ Multi Memory
- ▶ Premier Feedback Shield™
- ▶ Tri-Mode Layered Noise Reduction
- ▶ Dynamic Speech Enhancement
- ▶ Programmable Memories
- ▶ Memory Change Indicator
- ▶ Low Battery Indicator
- ▶ Power on Delay
- ▶ Wax Barrier
- ▶ Removable Concha Lock (R / L)
- ▶ Anti-Occlusion Loop (AOL)
- ▶ Low Group Delay

* Not all features may apply to your instrument.



PERSONA
MEDICAL

☎ 407 339 2422
📍 170 N. Cypress Way • Casselberry, FL 32707
✉ Info@PersonaMedical.com
🌐 PersonaMedical.com

UG / OLE / 3 © 11 Persona Medical

CE 0120



Welcome back to the world of hearing™

Welcome back to the world of sound!™

Congratulations. The Olé is one of the finest hearing instruments ever developed. The Olé hearing instrument is the first open fit, completely in the canal hearing device with AirTouch™ Multi-Memory. Amazing sound quality, noise reduction and virtual invisibility make Olé exactly what you desire from a hearing device.




Since 1967, Persona Medical has stayed on the cutting edge of innovation and technology. We thank you for allowing us to assist with your hearing needs.

These instructions along with the guidance from your Hearing Healthcare Professional will help you attain the maximum benefit from your Olé hearing instrument(s).



Identifying the Olé

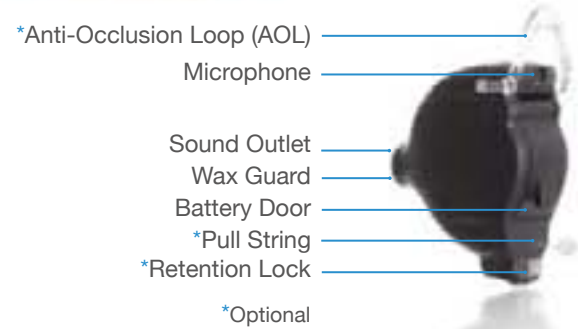

Identification



Every instrument has a unique serial number. The first two numbers indicate the year it was manufactured, and the remaining numbers are the serial number.

Inserting / Removing the Olé



Step 1 - Hold device by the Anti-Occlusion Loop (AOL) or the Retention Lock.

Step 2 - Make sure the AOL is up and the retention lock is down. Place the device in the canal with the sound outlet pointed towards the ear drum.

Step 3 - The AOL should prevent the top of the device from being inserted too deep into the canal. When the body of the device is placed correctly in the canal, push or place the retention lock into the concha (back of ear).

Step 4 - The Olé should be in the canal just behind the tragus with the battery door positioned vertically in the ear.

Removal - Remove from ear using retention lock or pull string.

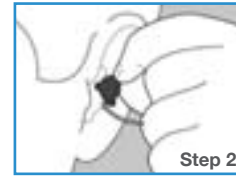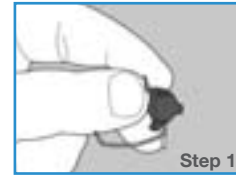

Powering the Olé

Batteries - The Olé uses a 10A Battery.

ON - With your finger, gently open the door. Insert battery and close the door. If your instrument is equipped with a power on delay, you will be given several seconds to seat the device comfortably in your ear before it turns on.



OFF - To turn your hearing instrument off, simply open the battery door wide enough so that the internal contacts are no longer touching the battery. Store in a safe dry place.

CAUTION: Batteries are dangerous if swallowed! Keep batteries out of the reach of children. Never put batteries in your mouth for any reason - they are easily swallowed. In the event a battery has been swallowed, seek immediate medical attention. For more information call the National Battery Hotline at 202-625-3333.


AirTouch™ Multi Memory

AirTouch Memory allows you to select between memories with the greatest of ease. Simply tap the outer part of your ear canal and the sound pressure triggers AirTouch. The Olé does not require buttons or switches and does not need to be touched to change memories.



The memories are preset by your Hearing Healthcare Professional to meet your hearing needs.

A few minutes of practice should be sufficient for you to become comfortable with AirTouch Technology.

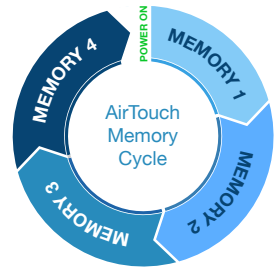

Patent Pending